

# Quick Guide cleaning



Cleaning the Surgical Kit



1 Take the surgical kit to the cleaning room



2 Set up the used drills and clean it properly



3 Place all drills in the cleaning liquid



4 All drills must be covered with liquid




5 Also place the metal bowl into the liquid



6 Put the lid in the liquid



7 The lid must be covered with liquid

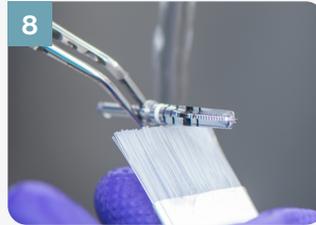

8 Clean the drills with a brush



9 Dry the drills



10 Put the drills on a dry cloth



11 Place all parts in the surgical kit



12 Wrap it for sterilization



13 Weld it air tight



14 Put it in the sterilizer



15 Start the sterilization program



# Mounting the Surgical Kit

