


# Patent™ Surgical Kit – Mounting Instruction

**The Surgical Cassette (SC0000) and the corresponding instruments are provided separately and unsterile.**

Before any surgery, the Surgical Cassette as well as all instruments have to be cleaned and disinfected separately according to the related IFU (MKT 020: Surgical Cassette; FB-EV 048: One-piece torque wrench and Wrench adapter; 120272 – Drill Bits:

Punches, Dental Drill Extension, Ankorn Drill, Pilot Drill, implant drills, thread cutters and guided countersinks; MKT 021: Direction indicator and implant drivers). Once all components are cleaned, disinfected and dry, the Surgical Cassette must be filled as described

in the figure below, placed in a pouch and sterilized. For each instrument, its specific position in the tray as well as the corresponding name and article number (in brackets) are indicated.

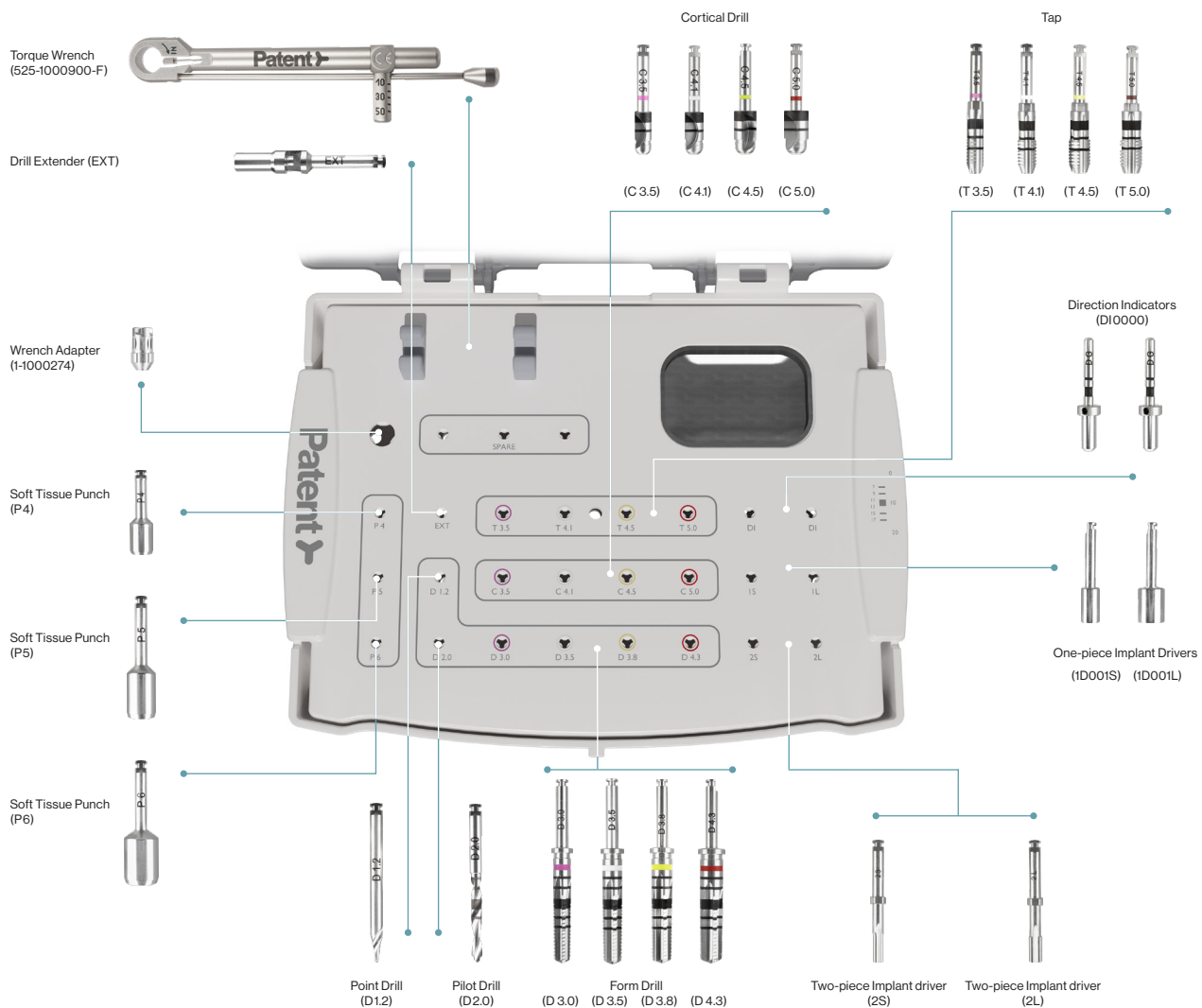

**Note:** Some items of the Patent™ Surgical Kit are not available in all countries.

**Zircon Medical Management AG**

Churerstrasse 66  
8852 Altendorf, Switzerland  
info@zircon-medical.com | www.zircon-medical.com



**ZV3 – Zircon Vision GmbH**

Hans-Urmiller-Ring 46c  
82515 Wolfratshausen, Germany